

Biopsy bags

IVD In vitro diagnostic – medical device


CND code: W05030580



General features

Small nylon bags for the processing of biopsy and histological specimens.

Code	Dimension	Packaging
07-00005	30 x 45 mm	1000 pcs.
07-00003	45 x 60 mm	1000 pcs.
07-00004	73 x 75 mm	1000 pcs.

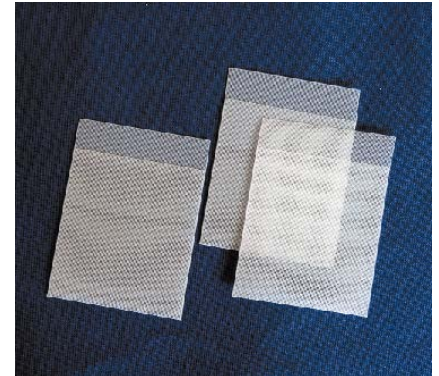


Manufacturer and exclusive distributor: Bio-Optica Milano S.p.A.

Features

Bio-Optica biopsy bags are produced by cutting and ultrasound solder; the thin mesh avoids the little sample's lost. The packing is constituted by 1000 bags in a plastic folder in a cardboard case.

For this kind of product, fixing validity date is not necessary, if the bags are properly preserved, far from direct light, heat and dust, at room temperature. Pore size: about 0,184 mm.


Method of use

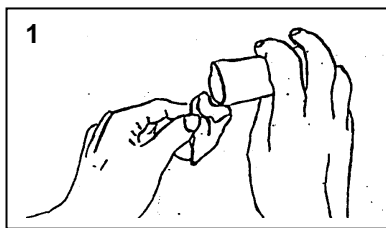

1. Empty the fixative and the specimen in the bag, the bag filter the fixative and the biopsy remain perfectly inside.

Close the bag and put it in a standard biopsy cassette.

Proceed as a usual processing.

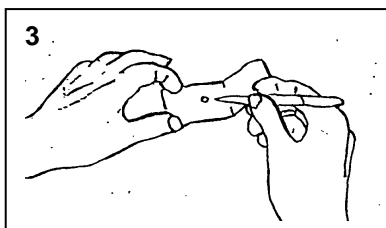

2. After processing extract the bag from the cassette, open it with care extending the edges.

3. Extract the specimen and embed it in paraffin.

