

Bio - Optica

Improving Pathology



Klessidra 3.0

No exposure

Work in safety without changing your habits

It doesn't reduce the exposure to formalin, it eliminates it!

In conformity with European Regulation 605/2014



1 Put the specimen into the empty container with or without the biocassette



2 Screw the prefilled formalin container on the container with the sample and rotate the lid in position O (open)



3 Tilt the device and let the formalin flow to the lower container



4 Rotate again the lid in position C (close)

PACKAGING :

- 90 ml of 10% neutral buffered formalin - capacity 12 cassettes.
Packaging 8 pcs. 05-01V125PK
- 160 ml of 10% neutral buffered formalin - capacity 2 Supermegacassettes or 20 cassettes.
Packaging 8 pcs. 05-01V250PK



www.bio-optica.it

Markkinoija Suomessa



Mediq Suomi Oy
Vuoritontunkuja 6, 02200 Espoo
puh. 020 112 1510
asiakaspalvelu@mediq.com
www.mediq.fi



Scan here to watch the video

Bio - Optica