

ACID PHOSPHATASE FIXATIVE

IVD In-vitro diagnostic medical device **CE**

CND Code: W01030705

Catalog number	Unit size
30-30120	1 x 100 ml

Packaging Primary container: amber bottle in polyethylene terephthalate (PET). Useful capacity 100 ml. PE cap. Tamper evident cap. Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol.

Expected aim Product for the preparation of cyto-histological samples for optical microscopy.

Application Fixative for acid phosphatase stain. Section thickness: 8 micron.

Method

1) Immerse sections in fixative	10 minutes
2) Air dry	
3) Incubate sections at 37°C in acid phosphatase (30-30118LY)	45 minutes
4) Wash in distilled water	10 minutes
5) Methyl green solution	5 minutes
6) Wash in distilled water	5 minutes
7) Mount with aqueous mounting media	

Components	Components	CAS	CE	Index
	Citrate Buffer 0.1 M pH 5.5	-	-	-
	Acetone	67-64-1	200-662-2	606-001-00-8

Warning and precaution The product must be used exclusively by specialized technical operators. Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risks presented by the mixture, the precautionary measures during use, the measures first aid and the intervention in the event of accidental release. Do not use if the primary container is damaged.

Storage Store the preparation at room temperature. Keep the containers tightly closed.

Stability After the first opening, the product is usable until the expiry date, if correctly stored. Product validity: 2 years.

Disposal Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.

Shipping The product can be shipped at room temperature

Date of issue: September 2020