LIFAair LMp800 FFP2-hengityssuojain

Käyttöohje

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The Lifa Air LMp800 FFP2 respirator meets the requirements of EU 2016/425 and standard EN 149: 2001 + A1: 2009.

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Takomotie 8, FI-00380 Helsingfors, Finland

Tel. 358 9 696 361

Conformity with the requirements for LMp800 FFP2 has been assessed by

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The respirator is designed to reduce the entry of airborne particles and liquid droplets into the airways. The respirator should not be used if the amount of airborne contaminants is more than 10 times the allowable amount.

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Store unused masks in the original package and protect from sunlight. Do not expose protective equipment to corrosive gases or other contaminants. Good ventilation. Ambient temperature -30 °C... + 40 °C, relative humidity <80%. Protect against light, heat and moisture during transport.

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Disinfect or wash your hands before touching the mask. Check that the package is not damaged and the mask and the head strap are not flawed. Check the date marking. Do not use the mask if the period of validity has expired.

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This product is not tested for use in a potentially explosive atmosphere. Do not use this product in areas with open flames. Facial hair may reduce the possibility to create a tight seal around the face of the user and thus reduce the protection efficiency. Do not use the product if the period of validity is expired or the product is visibly damaged. The product does not protect from gases or fumes. The product does not protect from oxygen deficiency or too high levels of oxygen. The product does not protect in situations where the contaminants or the risks they are causing are unknown. The respirator is designed in accordance with the marking NR (non reusable) for a maximum of one shift. Do not use the protector unless it is tightly sitting on the face.

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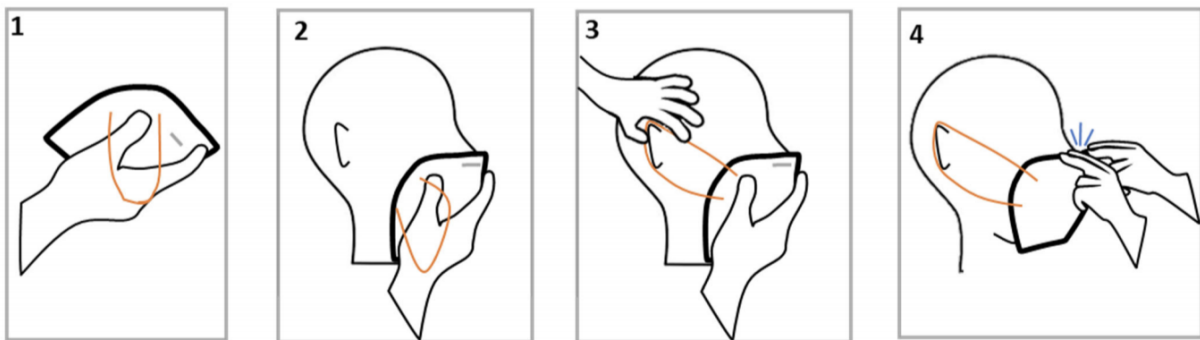
Use the mask continuously. Replace the mask to a new one if you need to remove the mask during your shift or if the mask is damaged. Disinfect or wash your hands before removing the mask. Remove the mask holding only the earloops and dispose of the mask safely. Disinfect or wash your hands after the disposal.

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With clean hands, open the respirator. Hold the respirator in one hand and let the earloops hang loose on the outside of the mask

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Place the respirator on your face with the nose clip on the bridge of your nose.

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Hold the respirator on your face with one hand. With the other hand pull the earloops over your ears.

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Using both hands, mold the nose clip firmly to fit the shape of your nose. Ensure proper seal between face and mask. Place both hands over the mask and exhale sharply. If air leaks from the sides or at the bridge of your nose, adjust the ear loops or nose clip, respectively, until the mask fits properly.

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