

# OSOM<sup>®</sup>

## RSV/Adeno Test



FOR USE WITH NASAL SWAB / NASAL SUCTION / THROAT SWAB (ADENO)

FEATURES:

- ✓ **ACCURATE DIAGNOSIS;**  
Results show 1 line for RSV and a separate line for ADENOVIRUS
- ✓ **SENSITIVITY** > 90% versus PCR for RSV; 85% versus PCR for Adenovirus
- ✓ **FAST;** Less than 1 minute hands on time; results in 10 minutes or less
- ✓ **EASY TO READ;** 2 color results, pink test lines and black control line
- ✓ **ROOM TEMPERATURE STORAGE**



TEST PROCEDURE / RESULTS

Add extraction buffer to test tube.

Add swab to tube and vigorously mix.

Place test stick into test tube.

Squeeze side of tube to express liquid.

Read results at 10 MINUTES.

**RSV POSITIVE**

**ADENOVIRUS POSITIVE**

**NEGATIVE**

ORDERING INFORMATION

- OSOM<sup>®</sup> RSV/ADENO TEST
- OSOM<sup>®</sup> RSV/ADENO CONTROL KIT

CONFIGURATION

25 TESTS

CATALOG NUMBER

197E (CE MARKED)  
198E (CE MARKED)

FOR EXPORT USE ONLY. NOT FOR SALE IN THE U.S.



**THE AMERICAS**  
Sekisui Diagnostics, LLC  
31 New York Avenue  
Framingham, MA 01701  
Phone: 800 332 1042  
Fax: 800 762 6311  
Email: osomrapidtests@sekisuidiagnostics.com  
ISO 13485:2003 Certified

**INTERNATIONAL**  
Sekisui Diagnostics (UK) Limited  
50 Gibson Drive  
Kings Hill, West Malling, Kent, ME19 4AF, UK  
Phone: +44 (0) 1732 220022  
Fax: +44 (0) 1732 220024  
Email: info@sekisuidiagnostics.com  
ISO 9001:2000 Certified, ISO 13485:2003 Certified



**Mediq Suomi Oy**  
Luomanportti 3, PL 115  
02201 Espoo, puh. 020 112 1500  
www.mediq.fi



1370-0 07/12

OSOM<sup>®</sup> and the OSOM Training logo are registered U.S. trademarks of Sekisui Diagnostics, LLC.  
©2012 Sekisui Diagnostics, LLC - All rights reserved.