

Super Mega cassettes


IVD In vitro diagnostic – medical device

CND code: W05030501



General features

Embedding cassette for macro-sections, new model
(see details at page 2).

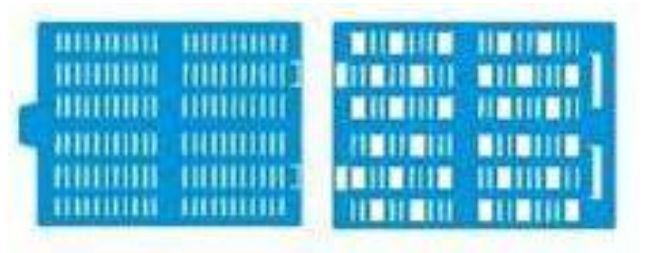

Code	Colour	Packaging
07-7000	White	200 pcs. (3 dispenser: 2 with 100 cassettes pcs. and 1 with 200 lids pcs.)

Manufacturer and exclusive distributor: **Bio-Optica Milano S.p.A.**

Technical features


Histological cassettes for the embedding of macro-sections, with a double closure system to guarantee a perfect lock-up during the processing passage.

Internal dimension: 62 x 48 x 14 mm, external dimension 73 x 53 x 16 mm.

The sample's identification is possible thanks to the wizen surface on three cassette's side. These surfaces are fitted to the writing with pencil. For a permanent marking, we suggest to use permanent marking pen Securline marker II Bio-Optica (code 11-51) or Histology pen (code 11-50).

These cassettes are compatibles with Bio-Optica filing system (Diateca for macro-sections and big archives).

NEW: beside traditional cassettes for macro sections, Bio-Optica Super Mega cassettes have some bigger holes. In this way, the paraffin hold on better to the cassette, during the embedding, avoiding block's break during the microtome cutting.

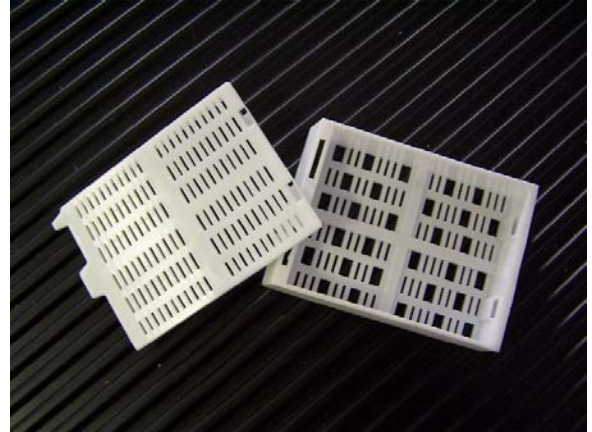

Functional features

Material: acetalic polymer (polyoximethylene polyacetal).

The acetalic polymer is stable to the high temperatures, to microwave operations, to solvents, to acids and alkalis; then this material is optimal for the processing and the embedding passages in the histological process.

The dispensers, used for packaging, are realized in cardboard with a frontal opening to ease the cassettes extraction.

For all these characteristics, particular storage precautions are not necessary for this kind of material.

Validity date: not applicable.

Note

Embedding cassettes are manufactured with our best technical skills, therefore, due to the complexity of the details themselves and the type of material used, is to be considered a physiological quantity of defective cassettes equal to 5,000 PPM (Parts Per Million), corresponding to a percentage of 0.5%.

Within this amount, Bio-Optica does not accept complaints and / or requests for warranty replacement.