

Bio - Optica

Improving Pathology



Klessidra 2.0

No exposure

Work in safety without changing your habits*

It doesn't reduce the exposure to formalin, it eliminates it!

In conformity with European Regulation 605/2014
*with atoxic buffer for an easy release of small biopsies



1 Open the container with the yellow lid and release the biopsy into the buffer



2 Screw the prefilled formalin container with the blue lid on the container with the biopsy and rotate the lid in position O (Open)



3 Tilt the device and let the formalin flow to the lower container



4 Rotate again the lid in position C (close)

PACKAGING :

- 10 ml of neutral buffered formalin 12% + 20 ml of phosphate buffer in container of 55 ml capacity
Packaging 27 pcs. 05-01V15PKF
- 10 ml of neutral buffered formalin 12% blue + 20 ml of phosphate buffer in container of 55 ml capacity
Packaging 27 pcs. 05-01V15PKFC

RACK:

- 16 places plexiglass rack for the transport of Klessidra
Packaging 2 pcs. 05-900900



www.bio-optica.it

Markkinoija Suomessa



Mediq Suomi Oy
Vuoritontunkuja 6, 02200 Espoo
puh. 020 112 1510
asiakaspalvelu@mediq.com
www.mediq.fi



Scan here to watch the video

Bio - Optica