

Metachromatic Crystal Violet

IVD In-vitro diagnostic medical device **CE**

CND Code: W01030708

Catalog number	Unit size
05-B31001	150 ml

Packaging Primary container: white bottle in High Density Polyethylene (HDPE). Useful capacity 150 ml. HDPE cap. Tamper evident cap.

Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol.

Expected aim Product for the preparation of cyto-histological samples for optical microscopy.

Application Stain for amyloid.

Method

1) Sections to water	
2) Metachromatic crystal violet	5 minutes
3) Acetic acid 0.5%	15 seconds
4) Wash in tap water	
5) Mount in aqueous mount	

Results Nuclei and cytoplasm.....blue-violet
Amyloidred-purple

Components

Components	CAS	CE	Index
Crystal violet	548-62-9	2089536	612-204-00-2
Acetic acid	64-19-7	2005807	607-002-00-6
Deionized water			

Warning and precaution

The product must be used exclusively by specialized technical operators.
Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risks presented by the mixture, the precautionary measures during use, the measures first aid and the intervention in the event of accidental release.
Do not use if the primary container is damaged.

Storage

Store the preparation at room temperature. Keep the containers tightly closed.

Stability

After the first opening, the product is usable until the expiry date, if correctly stored. Product validity: 1 year.

Disposal

Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.

Date of issue: May 2013