



People.Health.Care.

Suprasorb®

# CNP EasyDress Fixateur Externe **NEW**

For negative pressure therapy on extremities with external fixator.

Patented.



Suprasorb®

Controlled  
Negative  
Pressure



Suprasorb®

# CNP EasyDress Fixateur Externe **NEW**

An innovative seal that is easy to pull over.

1.20 m long




The **fourth** dimension.

The EasyDress Fixateur Externe adds a 4th dimension to the tried and tested EasyDress sizes S, M and L.



Universally applicable.  
With different external fixator models

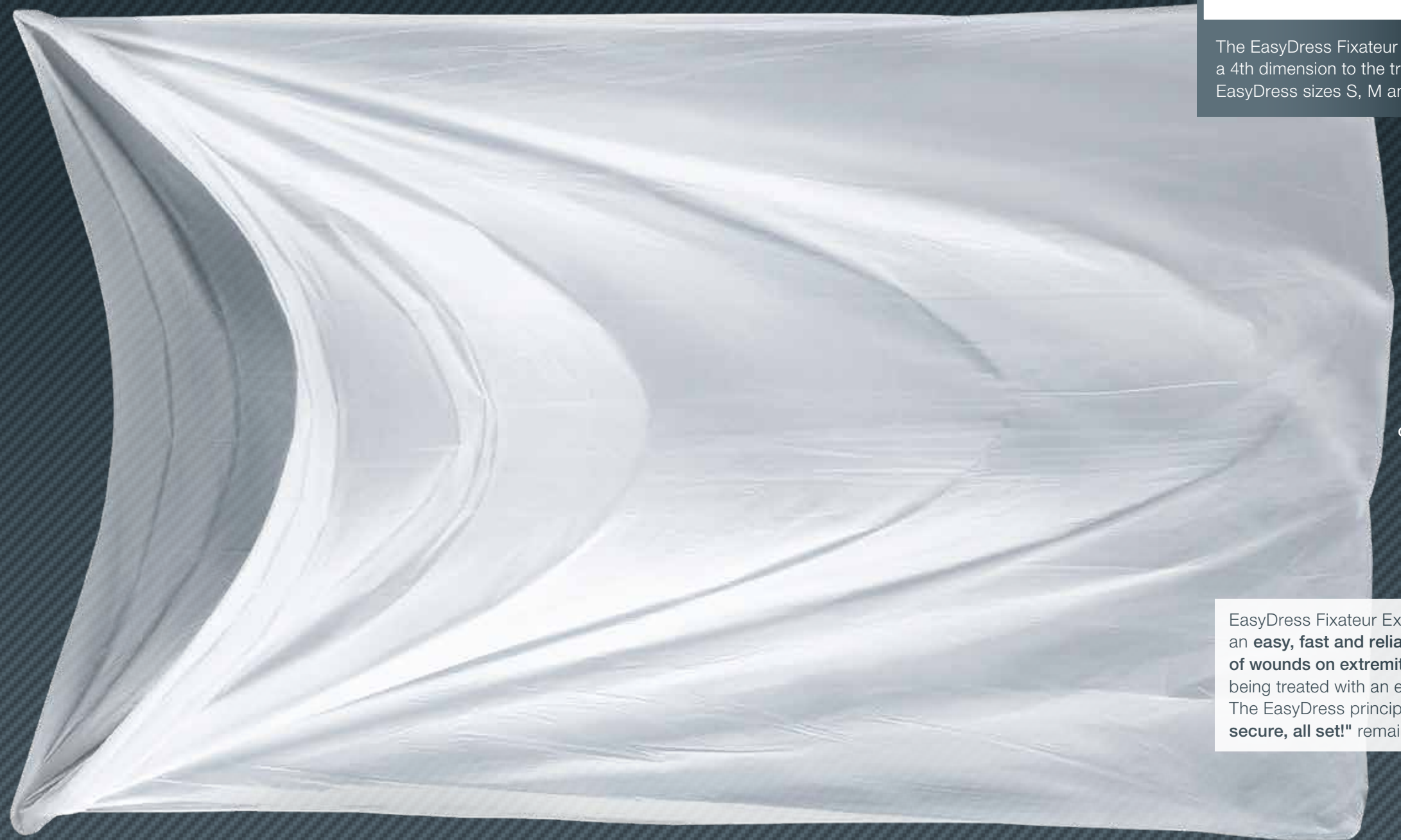

1.50 m  
circumference

EasyDress Fixateur Externe allows an **easy, fast and reliable sealing of wounds on extremities** which are being treated with an external fixator. The EasyDress principle "**pull over, secure, all set!**" remains the same.

# EasyDress Fixateur Externe

EasyDress Fixateur Externe is a secondary dressing for negative pressure therapy. For use on wound sites on extremities that are being treated with an external fixator.

- EasyDress Fixateur Externe is the new dimension in negative pressure therapy. It is 1.20 metres long and has a circumference of 1.50 metres.
- EasyDress Fixateur Externe allows for negative pressure therapy on extremities treated with an external fixator where it would not otherwise be possible.
- Thanks to its design an flexible special film, EasyDress Fixateur Externe becomes airtight considerably more quickly than conventional adhesive methods.



**CNP EasyDress Fixateur Externe** **NEW**

Application. Easy, fast and reliable.\*

# 1

 Preparation

- Treat the wound area with a wound filler approved for use in negative pressure therapy, e.g. wound foam.
- Cover pin entry points and intact skin under the external fixator, e.g. with an antimicrobial cotton gauze as shown here (Fig. 1).
- Fill in any spaces in the external fixator with wound foam. Wrap the edges with cotton gauze (Fig. 2).

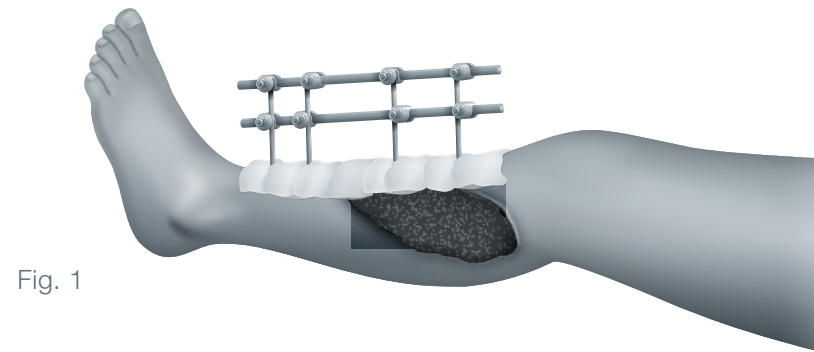


Fig. 1

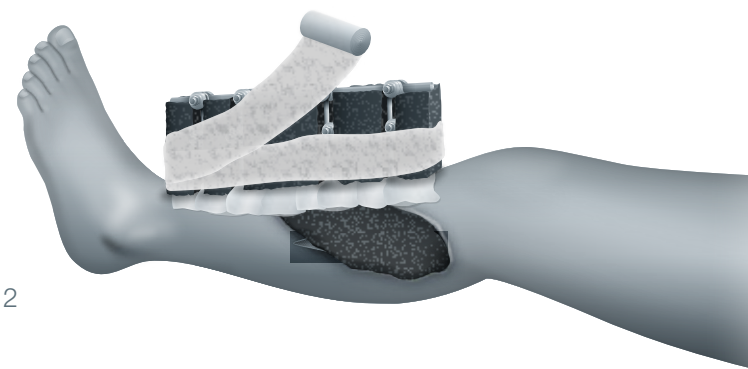


Fig. 2

# 2

 Pull over

- Pull the EasyDress Fixateur Externe over the extremity (Fig. 3).
- It should be positioned loosely over the external fixator (Fig. 4).

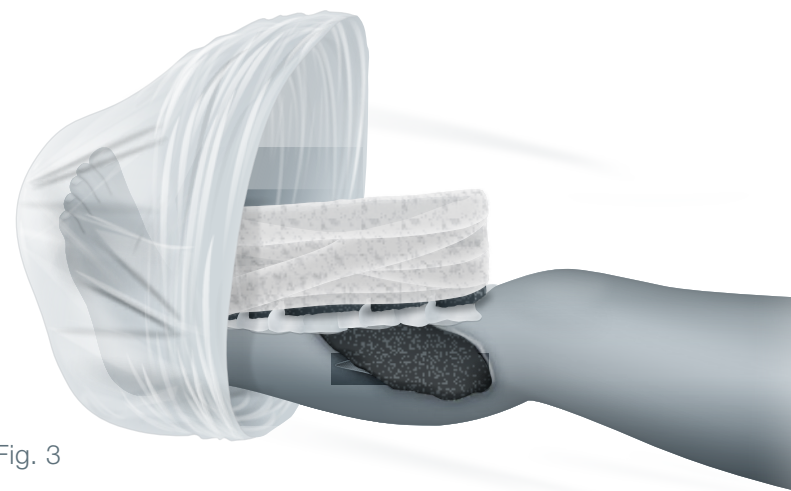


Fig. 3



Fig. 4

# 3

 Fix and seal

- Affix the EasyDress Fixateur Externe with retention strips placed circularly around the extremity (Fig. 5, no. 1).
- Seal off the rest of the open end of the EasyDress Fixateur Externe to form a bridge (Fig. 5, no. 2).
- Then fold the dressing and tape down to the extremity (Fig. 6).

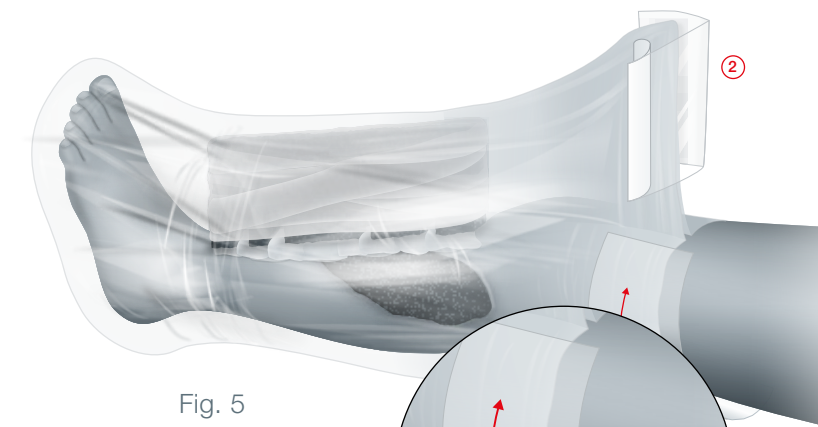


Fig. 5

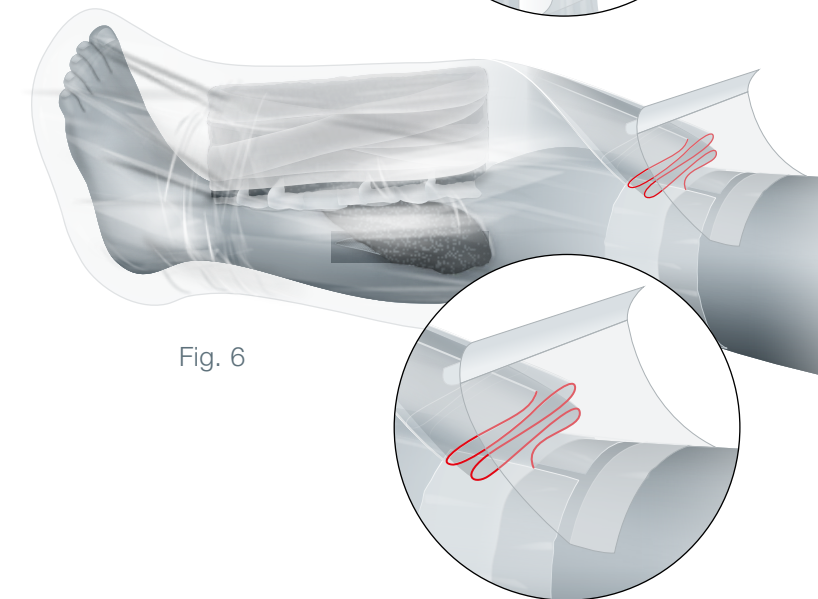
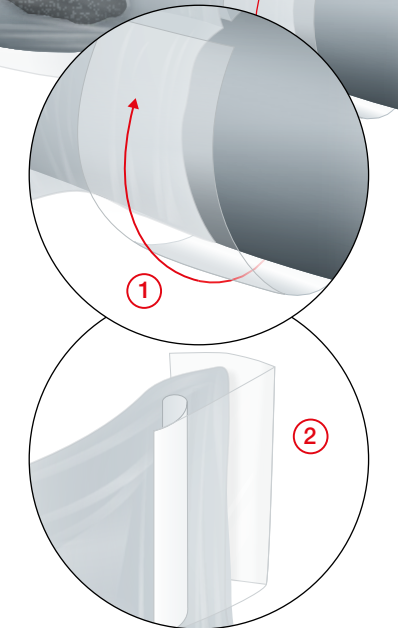


Fig. 6

# 4

 Start therapy

- Place the drainage pad over the hole cut in the EasyDress Fixateur Externe above the wound site and connect it to the device.
- Mould the EasyDress Fixateur Externe to the extremity and the fixator frame while negative pressure is being established.
- Start negative pressure therapy.



Fig. 7

\* The overview of the application procedure does not replace the instructions for use enclosed with the product.



Suprasorb®

# CNP EasyDress Fixateur Externe **NEW**

Germproof. Adaptable. Non-adhesive.



The material used for the EasyDress Fixateur Externe is identical to that used for the EasyDress range S, M, L. So it has been tried and tested over many years.



The special film for the EasyDress Fixateur Externe is airtight and adaptable. It is very well suited to modelling and sealing.



The high water vapour permeability of EasyDress Fixateur Externe ensures good moisture management underneath the dressing.




The material protects the whole surface from germs.



The non-adhesive EasyDress Fixateur Externe protects the intact surrounding skin. Quite the opposite of conventional adhesive methods.



The transparent special film makes it easy to monitor the primary dressing and the surrounding skin.

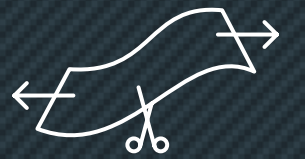

EasyDress Fixateur Externe is a reliable, tear-resistant secondary dressing. It can be cut to the desired length.

# EasyDress Fixateur Externe







People.Health.Care.

## Suprasorb® CNP EasyDress Fixateur Externe NEW

REF	Size	Tubular bandage		Retention strips
		1	2	10 cm x 25 cm
34905	One Size	150 cm	120 cm	6 pcs.

After sizes S, M, L the EasyDress Fixateur Externe is now the "4th dimension" in the EasyDress product range.

## Suprasorb® CNP EasyDress

REF	Size	Tubular bandage			Retention strips
		1	2	3	10 cm x 12 cm
34901	S	50 cm	30 cm	70 cm	5 pcs.
34902	M	65 cm	40 cm	85 cm	7 pcs.
34903	L	80 cm	50 cm	95 cm	8 pcs.



Suprasorb®